




IMPORTANT SAFETY INSTRUCTIONS. READ AND FOLLOW ALL WARNINGS AND INSTRUCTIONS BEFORE USING THIS PRODUCT. INJURY CAN RESULT FROM IMPROPER USE. SAVE THESE INSTRUCTIONS.

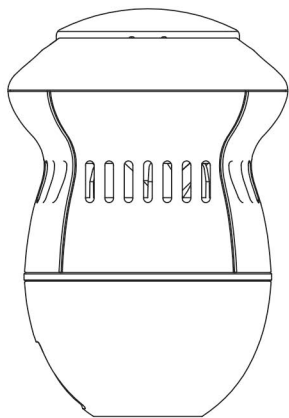

PRODUCT MAY VARY FROM IMAGES SHOWN

TABLE OF CONTENTS

Warnings.....	2
Product Components.....	3
How To Charge.....	4
How To Change Emery Pad.....	4
How To Use.....	5
How To Clean.....	6
Compliance.....	7

WARNING

STORE OUT OF REACH OF CHILDREN. THIS PRODUCT IS NOT A TOY.

THIS PRODUCT IS NOT WATERPROOF AND SHOULD NOT BE USED IN THE SHOWER OR ANY SITUATION THAT INVOLVES WATER. ELECTRIC SHOCK MAY RESULT. IF WATER ENTERS THE UNIT TURN OFF IMMEDIATELY AND STOP USE.

- Keep small plastic bags and packaging away from children to negate risk of choking.
- Do not store this product in the bathroom or a room that has high moisture content. Moisture can reduce the performance of the Pedi Vac™ and/or cause the unit to malfunction.
- Do not operate in the presence of explosives and/or other flammable fumes or liquids.
- Do not store or use this product near sources of heat or fire.
- Do not leave Pedi Vac™ charging unattended as this may impair the battery life.
- Intended for use on callused feet only. Do not use on uncalledus, smooth skin. Injury may result. Avoid use on irritated, inflamed or infected skin, or on open wounds. Irritation or lacerations may result. Avoid use if you are diabetic or have poor blood circulation. Consult a physician before use. Do not use on areas of the body that have undergone any sort of surgery or have any scars.
- If you experience pain while using this product, discontinue use immediately.

NOTICE



- Do not insert any foreign objects into the unit to avoid damage to the unit.
- Do not put this product in washing machine or dishwasher to clean to avoid damaging the unit.
- Turn off Pedi Vac™ when not in use to preserve battery life.
- Unplug the unit when charging is completed to preserve battery life.

PRODUCT COMPONENTS

A. Pedi Vac™ Unit
B. Emery Pad Cover
C. Emery Pad
D. Charging Port
E. Power Button & Low/High Selection
F. Battery Indicator & Speed Light
G. Circular Sponge Filter
H. Emery Pad Remover
I. Buffing Pad
J. Emery Pad Cleaning Brush
K. USB Charging Cable



HOW TO CHARGE

1. Turn the Pedi Vac™ on to check the battery by holding down the power button for 2 seconds. Battery indicator light is solid green when fully charged. When indicator light flashes yellow battery is low.
2. To charge, plug the micro-USB connector end of the supplied USB cable into the charging port of the Pedi Vac™ (Figure A). Plug the other end of the USB cable into a USB wall adapter, then plug the adapter into the wall outlet.
3. While charging the red light will be solid, indicating the Pedi Vac™ is charging properly. When charging is complete the light will revert back to solid green (Figure B). Once charging is complete, battery indicator light will turn off. To check battery level, click the power button once while unit is still plugged in.

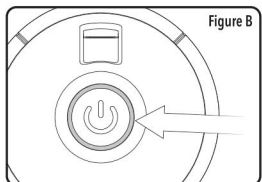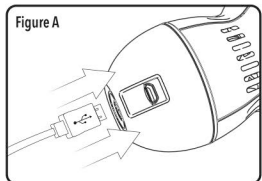

NOTE: Make sure to remove the USB cable from the charging port of the Pedi Vac™ and the USB port of the power source after the Pedi Vac™ is fully charged. Over-charging the battery can cause poor battery life.


HOW TO CHANGE EMERY PADS

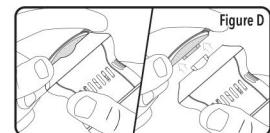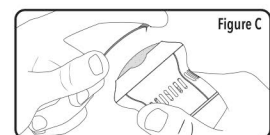


1. Place one hand on the unit and in the other hand hold the Emery Pad Remover (Figure C).
2. Place both ends of the emery pad remover underneath the rims of the emery pad and gently lift upward (Figure D). The emery pad will make a slight click sound to indicate proper removal.
3. You may use the buffing pad to further smooth down your feet, or to maintain their appearance in between treatments with the Pedi Vac™.

Page 4

HOW TO USE

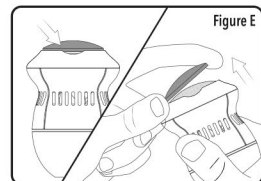

1. Start by removing the emery pad cover by lifting up on one side (Figure E).
3. When unit is in the on-position speed is low. To achieve high speed, click the power button once again.

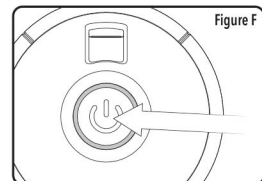


2. Turn on Pedi Vac™ by holding down the power button for 2 seconds (Figure F).

NOTE: The LED light will illuminate green when unit is in low speed mode. To achieve high speed, click the power button again and the LED light will change to solid red.

To turn off Pedi Vac™ hold down the power button until light turns off.

WARNING: Do not immerse the unit in water or allow water anywhere near the unit. Pedi Vac™ is NOT water proof. Store in a dry place. Such measures reduce the risk of electric shock and damage.

4. Holding the Pedi Vac™ securely, gently press the emery pad to the thickest area of dead skin you want to remove and slowly move outwards towards healthy skin (Figure G).

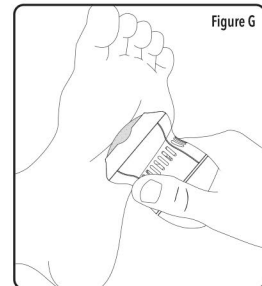


5. Check intermittently for desired results. Once new, healthy skin is revealed or callus is removed, stop use and switch the Pedi Vac™ off.

Page 5


HOW TO CLEAN

1. Start by removing the emery pad following instructions *How To Change Emery Pad*. Emery pad can be cleaned by running under warm water. (Figure H)

Allow the emery pad to air dry. Do not attempt to clean the main unit by running under water. To clean the main unit wipe down with a soft damp cloth.

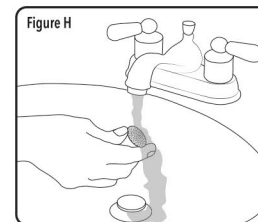


2. Once emery pad is removed turn the Pedi Vac™ cover counter clock wise to open shavings chamber (Figure I).

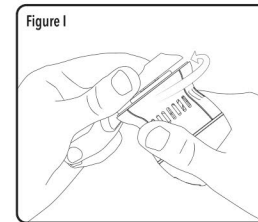

3. Slowly remove circular sponge filter from chamber. Rinse filter with warm water and allow to air dry.

NOTE: Circular Sponge Filter and Emery Pad can be used up to 100x until replacement is needed.

4. Clean the remaining skin shavings out of the chamber with the provided shavings remover brush (Figure J).

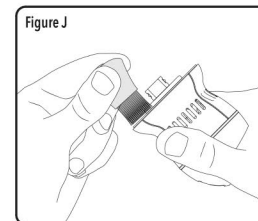

5. Once chamber is fully cleaned replace the circular filter, shavings chamber cover & emery pad securely.

NOTE: For best performance, clean PediVac™ after each use.

Page 6

This AC wire carrier current device of unintentional radiator is a Category II equipment compliant with Industry Canada's ICES-006.

LIMITATION OF LIABILITY

Liability limited to the purchase price of this product. Supertek Canada Inc. shall not be liable for any incidental or consequential damages or any implied warranty on this product.

Clikit.ca
Distributed by
Supertek Canada Inc.
Mont-Royal, QC H4T 1X2
www.clikit.ca
Customer Service:
1-800-304-8354

Page 7