

WRIST RELIEF PLUS

Visit www.clikit.ca for the full collection.

HOW TO USE

LEFT HAND:




- The wrist sleeve comes packed & ready to be worn on the left hand.
- Put sleeve on your hand. Insert your thumb into the thumb hole. The thumb hole should be facing the right side.
- The Copper Fit logo should be facing up and your palm should be resting on the beaded cushion (Image B).

RIGHT HAND:

- Remove the flexible stabilizer from the pocket (Image C).
- Turn the sleeve inside out. The thumb hole should be facing the left side.
- Put sleeve on your hand. Insert your thumb into the thumb hole.
- The Copper Fit logo should be facing up and your palm should be resting on the beaded cushion.

To remove the flexible stabilizer, locate the pocket and pull the flexible stabilizer out (Image C).

To insert the flexible stabilizer, locate the pocket and insert the flexible stabilizer until secure (Image E).

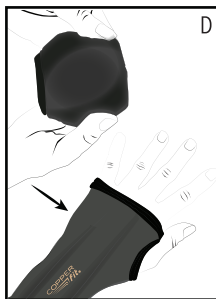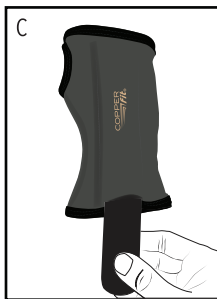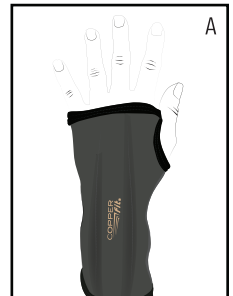

LIMITATION OF LIABILITY

Liability limited to the purchase price of this product. Supertek Canada Inc. shall not be liable for any incidental or consequential damages or any implied warranty on this product.

Distributed by:

SUPERTEK CANADA Inc., Mont-Royal QC H4T 1X2

WWW.CLIKIT.CA

Customer Service: 1-800-304-8354 • For more information, visit copperfitproducts.ca