



Thank you and congratulations on your purchase of Miracle Copper™ anti-Fatigue Compression Socks. The socks with *Graduated Compression* and *Copper Infused Fibers* to energize your tired feet & legs!

DIRECTIONS

- Insert hand into sock, pinch heel and partially pull sock inside out. (Fig. 1)
- Place sock over toes (Fig. 2) then pull sock up to heel and align so heel fits comfortably. Leg section of sock should still appear inside out.
- Slowly pull up leg section of sock until it sits just above your calf (Fig. 3). Adjust around calf until comfortable.
- Repeat Steps 1-3 on opposite foot.

CARE & CLEANING

- Hand Wash or Machine Wash Cold, Delicate Cycle
- Wash with Like Colours
- DO NOT BLEACH
- Air Dry or Tumble Dry Low

83% Nylon, 10% CuTEC® Copper Nylon,
7% Lycra® Spandex

Distributed by:

IVSupertek™

Mont-Royal, Qc H4T 1X2
client@supertek.ca

Customer Service: 1-800-304-8354

FIG. 1

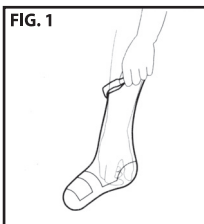


FIG. 2

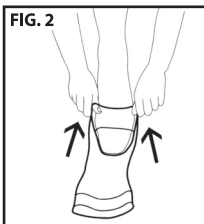


FIG. 3

