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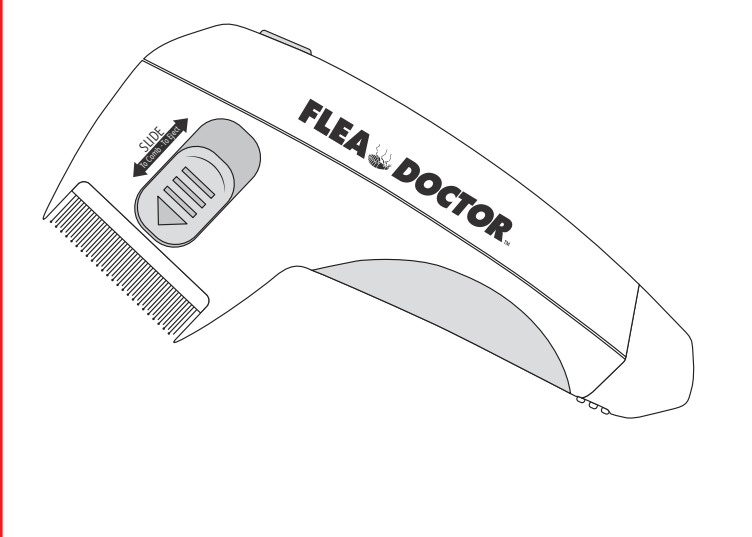
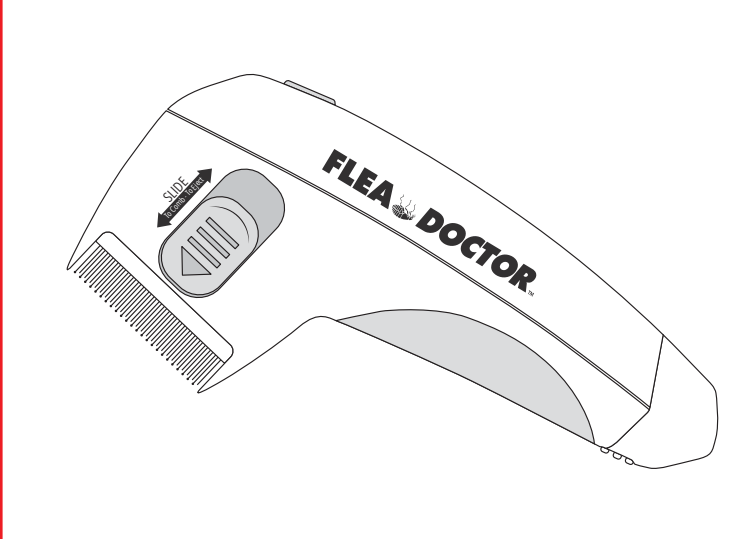
Kills Fleas* Without Using Pesticides
*80% stunned or killed on contact

IMPORTANT SAFETY INSTRUCTIONS, READ AND FOLLOW ALL WARNINGS AND INSTRUCTIONS BEFORE USING THIS PRODUCT. INJURY CAN RESULT FROM IMPROPER USE. SAVE THESE INSTRUCTIONS.

PRODUCT MAY VARY FROM IMAGES SHOWN

TABLE OF CONTENTS

Warnings 2
Product Components 3
Battery Installation & Replacement 3
Getting Started 4
How To Use 4,5,6
How To Clean 6

INTRODUCTION

The Flea Doctor™ is designed to stop and remove adult fleas from your cat or dog. The Flea Doctor™ uses a small electrical charge to kill or stun 80% (average) of the fleas that contact the comb's metal teeth. Actual results will depend on the combing efforts of the user and contacting the fleas with the comb. The Flea Doctor™ can be a helpful part of your home flea control program. Regular, thorough combing with the Flea Doctor™ removes adult fleas from your pet. To help avoid reinfestation you also must remove flea eggs and larvae which will be found in carpets, furniture and pet bedding. Regularly vacuum carpeting, upholstered furniture, low hanging curtains and wood floor cracks to remove eggs and larvae. Regularly wash pet bedding in hot soapy water.

NOTICE

- DO NOT store or use this product near sources of heat or fire as these may damage the unit.
- DO NOT immerse Flea Doctor™ in any fluid or liquid as this may damage the unit.
- DO NOT mix old and new batteries. For best performance, when replacing batteries REPLACE ALL TWO (2) BATTERIES at the same time.
- DO NOT mix alkaline, standard (carbon zinc) or rechargeable (nickel-cadmium) batteries. Use only the size and type of battery specified. Mixing batteries of different type, ages or brands may cause them to leak or rupture.

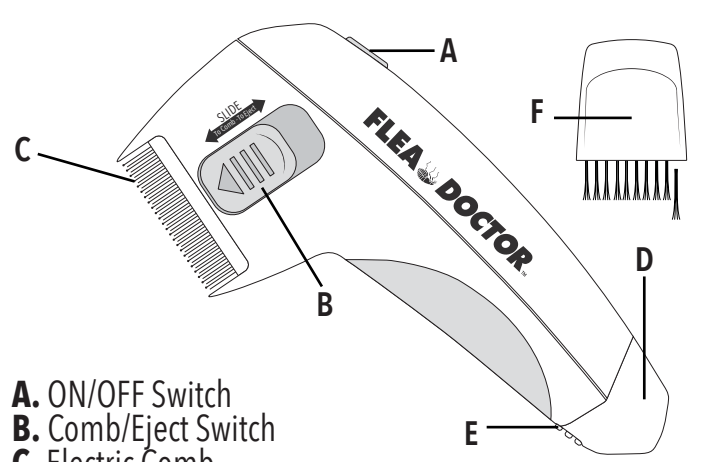
WARNING

- **STORE OUT OF REACH OF CHILDREN. THIS PRODUCT IS NOT A TOY.**
- **SUFFOCATION/CHOKING HAZARD**: To avoid danger of suffocation or choking, keep plastic bags away from babies and children.
- DO NOT operate in the presence of flammable fumes or liquids or explosive materials. Flea Doctor™ generates small electrical arcs that may ignite any flammable vapors that are present.
- **CONSULT YOUR PHYSICIAN** prior to using the Flea Doctor™ if you suffer from epilepsy, or have a pace-maker or other neurostimulator.

CAUTION

- Electrical shock hazard -
 - DO NOT immerse Flea Doctor™ in any fluid or liquid. This product is not waterproof or water resistant. Do not use if it has become wet.
 - DO NOT use Flea Doctor™ when your pet's hair is wet or damp. Make sure your pet's hair is completely dry before use.
 - DO NOT attempt to open the housing of Flea Doctor™ under any circumstance.
- DO NOT touch the metal teeth of the Flea Doctor™ while operating; touching teeth may cause uncomfortable tingling sensation. Flea Doctor™ is not designed for use on people and may cause injury.
- DO NOT run Flea Doctor™ near your pet's wounds, scars, warts, bruises, rashes or moles. The firm metal teeth may cause pain or injury to your pet.
- DO NOT drop the Flea Doctor™, as the unit or its parts may break or suffer damage, rendering it unsafe for use.
- Inspect the Flea Doctor™ prior to use. If there are visible signs of damage, DO NOT turn the unit on.

PRODUCT COMPONENTS

- A.** ON/OFF Switch
B. Comb/Eject Switch
C. Electric Comb
D. Battery Compartment
E. Battery Compartment Release Button
F. Cleaning Brush

BATTERY INSTALLATION & REPLACEMENT

1. Open the Battery Compartment cover. (Figure A)
CAUTION: Ensure unit switch is in the OFF position before installation.
2. Insert 2 AA batteries (not included) into the Battery Compartment. Make sure to line up the (+) and (-) markings on the battery with the (+) and (-) markings on the inside of the Battery Compartment. (Figure B)
3. Reinstall the battery cover.
NOTE: Batteries should be replaced if the unit does not make a sound when turned on, or if you detect a drop in performance. See **NOTICE** section (Fig. 1) for proper procedures.

GETTING STARTED

First choose where to comb your pet. Combing outside is best to prevent any live fleas that escape from populating your home. If combing indoors, do it on top of a newspaper or any type of disposable floor covering to catch any fleas or debris that may drop off during the combing process.

Before combing with the Flea Doctor™, brush your pet with a regular pet brush to remove twists and tangles in the fur that may interfere with thorough combing, particularly if your pet's fur is thick, long or knotted. **DO NOT** use Flea Doctor™ when your pet's hair is wet or damp. Make sure your pet's hair is completely dry before use. Before combing your pet with the Flea Doctor™, prepare a bowl or bucket of warm water and dish soap mixed together. This will serve as a receptacle for the fleas and other debris removed by combing. Any fleas not killed by the comb will be killed in the soapy water.

HOW TO USE

1. Start by releasing the Electric Comb teeth outward by pushing the "Slide to Comb/Eject Switch" forward. (Figure C)
2. Once the Electric Comb teeth are released, turn the Flea Doctor™ on by pushing the ON/OFF Switch upward into the ON position. (Figure D)
NOTE: Successful use of the Flea Doctor™ depends on the user's good combing practices.

HOW TO USE

3. Keep your pet calm and in the standing position. Place one arm around the neck or abdomen to hold your pet still without choking, and hold the Flea Doctor™ with your free hand. Working slowly and gently, stroke the comb in the natural direction of the fur growth to remove adult fleas. Begin combing at your pet's head, neck and shoulders, and work towards the back and tail. For large or fidgety pets, it may be easier to comb with two people.

NOTE: Hold the Flea Doctor™ metal teeth at a 45-degree angle to the surface of the pet. (Figure E) Reduce pressure if your pet appears to be in any discomfort from the Flea Doctor™.

4. The comb may quickly fill up with fleas, bits of fur, and other debris. This debris on the comb may make the Flea Doctor™ less effective by preventing other live fleas from contacting the metal teeth. Watch for the buildup of debris while combing and remove it from the teeth frequently by first turning it off, and then pushing the "Slide to Comb/Eject Switch" backwards and forwards to remove debris from the teeth. Drop all debris into the bowl of soapy water. (Figure F)
NOTE: Unit will make a beep sound every few seconds indicating it is on and working. Make sure the comb teeth are clear of fleas and debris after each sweep through your pet's fur. Flea Doctor™ will continue to beep rapidly until comb teeth are cleared.

HOW TO USE

5. Continue combing down the back, chest, sides and belly of your pet. Finally, finish on your pet's legs, paws and tail. Be sure to focus on the lower back area above the tail where fleas may tend to congregate.
IMPORTANT! DO NOT comb over your pet's sensitive areas such as eyes, ears, mouth & genitals as this may annoy or injure your pet. Also avoid combing over any wounds, scars, warts, bruises, rashes or moles. The firm metal teeth may cause pain or injury to your pet.
6. Make sure to comb your pet's whole body. Comb as close to the skin's surface as possible to ensure optimal flea removal. Fleas may be missed as they move deeper into the fur. A second brushing is recommended a few minutes later to catch any fleas missed.
NOTE: Breeds with thick fur will require greater care to locate all fleas.
7. Once you've completed the flea combing session, let the soapy water stand for 10-15 minutes so any live or stunned fleas drown in the bowl. Then flush the soapy water and discard any newspaper or disposable floor coverings in an outdoor receptacle or in a sealed trash bag.

HOW TO CLEAN

1. Clean the metal teeth of the Flea Doctor™ during use as needed and after each combing. With the provided Cleaning Brush, brush out any remaining debris or fleas from the Flea Doctor™ Electric Comb. (Figure G) If cleaning indoors, brush debris into the bowl of warm soapy water to kill any live or stunned fleas. See "Getting Started."
2. To clean the exterior of the unit, wipe down with a clean damp cloth.

LIMITATION OF LIABILITY
Liability limited to the purchase price of this product. Supertek Canada Inc. shall not be liable for any incidental or consequential damages or any implied warranty on this product.

Distributed by:
Supertek Canada Inc.
Mount Royal, QC H4T 1X2
Customer Service:
1-800-304-8354
www.clikit.ca