



Instructions For Use

Material Contents:
Neoprene and nylon

Putting On the Ankle Genie™ for First Use:

1. For first use, unfasten the hook and loop tabs, keeping the zipper closed.
2. Slide your foot through the arch support.
3. Reattach hook and loop tabs, adjusting to your desired level of compression without over tightening (Figure 1).

NOTE: Ankle Genie™ can be worn over socks and in shoes if desired.



Removing the Ankle Genie™:

1. To remove, use the zipper to open the Ankle Genie™.
2. Slide foot out, leaving the hook and loop tabs attached for next use. After first use, Ankle Genie™ can be put on using only the zipper, adjusting the tabs only if you require more or less compression than the previous use (Figure 2).



CAUTION: If foot or ankle pain persists or increases, discontinue use and consult a physician. Some individuals may be sensitive to neoprene rubber. If you notice skin irritation, discontinue use and consult a physician.

Care Instructions:

Close all fasteners, hand wash and air dry. Do not bleach.
Do not iron.

LIMITATION OF LIABILITY

Liability limited to the purchase price of this product.

TeleBrands Corporation shall not be liable for any incidental or consequential damages or any implied warranty on this product.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

TELBrands®

Fairfield, NJ 07004
www.Telebrands.com
Made in China.

©2013 Telebrand Corp.

TBAGIM121913